
Rebroadcasting from Frontrow JUNO to Roger Pen



- Using a 3.5mm jack to jack lead connect to the AUDIO OUTPUT from the JUNO,
- With the other end of the lead connect the Roger pen transmitter using the AUDIO LEAD (Mini USB to jack) provided,
- Switch on the Roger system and UNMUTE the Juno microphone,
- Whilst talking into the JUNO microphone increase the AUDIO OUTPUT control to around volume 6.5,
- Ensure the hearing instrument user can hear the JUNO microphone but also the environmental sounds via the hearing aid.

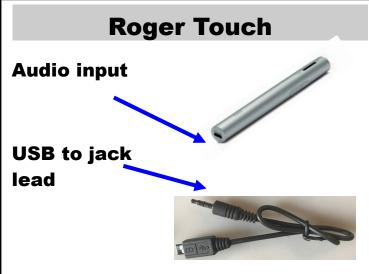



01823 323363 info@robertsaudiosolutions.co.uk, www.robertsaudiosolutions.co.uk

